- Ⓛ Gebrauchsanleitung
- Ⓜ Instructions for use
- Ⓝ Mode d'emploi
- Ⓔ Instrucciones para el uso
- Ⓜ Istruzioni per l'uso
- Ⓝ Gebruiksaanwijzing

- Ⓛ Instruções de utilização
- Ⓜ Brugsanvisning
- Ⓝ Bruksanvisning
- Ⓔ Instrukcja obsługi
- Ⓜ Οδηγίες χρήσης
- Ⓝ Návod k použití

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Ⓜ Hand activity cushion

This cushion is suitable for use in occupational therapy as well as for promoting motor skills, activating the senses and improving concentration. It can also be used for agitation, tension and restless hands. The front and back of the cushion cover have different elements to activate the senses. People with limited motor skills or severe agitation may discover a calming occupation in the cushion.

Please note: The inner cushion is vacuum-packed: Open the zip of the outer cover, take out the vacuum-packed inner cushion and carefully remove the film. Leave the cushion for several hours until it has reached its full size. A mild smell of the product when new is normal. The smell automatically vanishes; if needs be, the product can be ventilated in the fresh air.

Cleaning: The outer cushion cover may only be spot cleaned by hand. For this purpose, clean the soiled areas with a damp cloth (if necessary with mild soapy water) and allow to air dry. The inner cushion is washable according to the instructions on the care label.

Caution: This product is not a toy! The cushion is not suitable for children under 6 years of age. Pulling too firmly may cause small parts to come loose; risk of suffocation.

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