④ Gebrauchsanweisung
④ Instructions for use
④ Instrucciones para el uso
④ Mode d'emploi
④ Istruzioni per l'uso

④ Gebruiksaanwijzing
④ Instruções de utilização
④ Brugsanvisning
④ Bruksanvisning
④ Instrukcja obsługi

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- 1 x Thermo-Stoffmanschette
2 x Gelkissen zur Kalt-/Warmanwendung

④ Thermal Wrap / Thermal Belt

The Thermal Wrap / Thermal Belt is suitable for hot and cold application. Therefore, use the supplied gel pack and put it into the freezer or the microwave or into hot water. Please follow the instructions below!

Cold application:

- 1) Refrigerate the gel pack for at least 2 hours before use
- 2) Put it into the supplied fabric cover
- 3) Apply the cold pack for a maximum of 15 - 20 minutes at a time
- 4) Take a break of at least 20 minutes before next use

Do not apply directly to skin!

Heating in the microwave:

Since the actual performance of various microwave oven manufacturers with the same specification is different, we recommend testing the optimal warm-up time.

- 1) Place the gel pack into the microwave oven and distribute the gel evenly
- 2) Turn on the microwave oven on the highest setting for at least 30 seconds
- 3) Allow the gel pack to rest for a minute and remove it and knead it well so that the warmth is evenly distributed
- 4) Eventually repeat the procedure in steps of 10 seconds, until the desired temperature is reached. Put the gel pack into the fabric cover before use.

Hot water method:

- 1) Boil approximately 4-5 litres of water
- 2) Remove the water from the heat source
- 3) Immerse the gel pack in the water for approximately 10 minutes
- 4) Remove the pack carefully from the hot water and put it into the fabric cover before use.

Warning: Caution! Risk of scalding!

Attention: Overheating can destroy the gel pack. Leaky or empty gel packs must not be heated. Do not leave the gel pack unattended while heating!

Gel information: The contents of the gel pack are non-toxic. Should gel come into contact with the skin or eyes, simply wash off with plenty of water. There is no record of skin or eye irritations. In order to ensure enjoyment of your new product for the longest possible amount of time, you should follow these instructions exactly!

Keep out of reach of children!

Included in delivery:

- 1 x fabric cover
2 x gel packs for cold and hot application

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