- Ⓒ Gebrauchsanweisung
- ⒸⒹ Instructions for use
- Ⓕ Mode d'emploi
- Ⓔ Instrucciones para el uso
- Ⓘ Istruzioni per l'uso
- Ⓝ Gebruiksaanwijzing

- Ⓔ Instruções de utilização
- ⒸⒹ Brugsanvisning
- Ⓕ Bruksanvisning
- Ⓔ Instrukcja obsługi
- ⒸⒹ Οδηγίες χρήσης
- Ⓔ Návod k použití

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ⒸⒹ Digital hearing aid with rechargeable battery

1. Device description

- Ⓙ Silicone earplugs (4 types)
- Ⓒ Volume control
- Ⓒ Power ON/OFF
- Ⓒ Microphone
- Ⓒ LED battery indicator
- Ⓒ Charging port
- Ⓒ Charging point
- Ⓒ LED indicator
- Ⓒ Charging port (cable)

2. Use

Select a suitable silicone earplug. The earplugs can be removed and replaced. Minimise the volume before using the device. Switch the device on at the ON/OFF switch and insert it into the desired ear (right or left). Position it in such a way that it fits firmly and comfortably in the ear. It can also be turned slightly in the ear. Switch the hearing aid off after each use and recharge if necessary.

3. Volume

The volume can be adjusted by means of the control wheel.

- Clockwise: loud +
- Anticlockwise: soft -

4. Charging procedure

Use the supplied cable and a suitable adapter (e.g. from a smartphone/mobile phone) or other USB connector. Connect the cable to the charging station. The LED lights up red *.

Switch the hearing aid off and place it in the charging station. Make sure that the hearing aid is in contact with the charging station.

The charging indicator lights up blue during the charging procedure, and changes to green when the battery is fully charged. Charging takes about 1 hour. Up to 30 hours battery operation.

5. Safety instructions

- Keep the device away from moisture and high temperatures (>40°C).
- Keep the device away from X-rays and magnetic fields.
- Keep the hearing aid in a dry and safe place
- Minimise the volume prior to insertion and only then adjust it to the desired level
- Make sure to increase the volume gradually to avoid the sudden amplification of sounds.
- The device is NOT suitable for children.
- This device is designed to enhance your hearing capacity and is not a medical hearing aid. It is primarily intended to assist in the case of minor hearing impairment and for use as a replacement device.
- Please consult a specialist if you are experiencing problems with your hearing.

6. Technical data

Weight: 3 g
 Voltage: 3.7 V DC
 Frequency: 300-5000 Hz
 Output volume: ≤ 128 dB + 3dB
 Input volume: ≤ 29 dB + 3dB
 Acoustic amplification: 28dB +/- 5dB
 INPUT: 100V~240V, AC 50/60Hz, 0.2-0.25A
 OUTPUT: 5V 1A

Batteries, rechargeable batteries and electrical devices must not be disposed of with the normal household waste! All

consumers are legally obliged to submit all batteries, rechargeable batteries and electrical devices, whether they contain pollutants or not, to a collection point in their community or district or to return them to a dealer, so that they may be disposed of an environmentally compatible way. Only submit batteries and rechargeable batteries in a discharged state!

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