SEEPLUS

SEEPLUS ADJUST


-6.0D ↔ +3.0D

INSTRUCTIONS FOR USE / INSTRUCCIONES DE USO / GEBRAUCHSANWEISUNG
MODE D'EMPLOI / ISTRUZIONI PER L'USO / BRUKSANVISNING


GL01

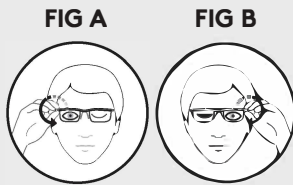49mm □ 22mm, 141mm PD:63mm

SEEPLUS
ADJUST

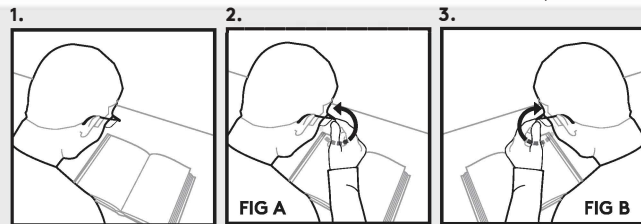EN 1. Place reading material on a flat surface.
2. Close one eye and adjust the focus by turning the dial located on the temple.
3. Close the other eye and turn the dial.

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
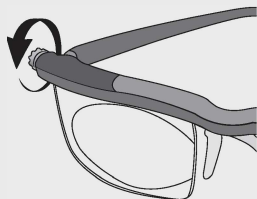
SEEPLUS
ADJUST

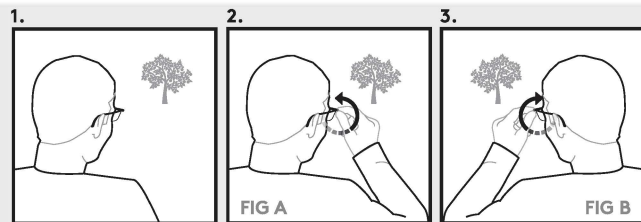EN 1. Look at a distant object (i.e. TREE or Television).
2. Close one eye and adjust the focus by turning the dial located on the temple.
3. Close the other eye and turn the dial.

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EN

ADJUSTABLE FOCUS GLASSES

Sliding lenses change the focal power by the turn of a dial.

- Have your eyes examined by an Eye Care Professional.
- Eye exams can detect serious health problems that have no symptoms.
- Adjustable eyewear is not intended to replace prescription eyewear.

- **DO NOT** use as eye protection.
- **DO NOT** wear if discomfort is caused.
- This device is **NOT** intended to correct astigmatism.
- **NOT** for permanent use.

- **STORE** this device in its protective case when not in use.
- **USE** a damp lens cloth to clean the lenses.
- **DRY** the lenses with clean cloth.
- **CLEAN** with care.
- **DISPOSE** of device in accordance with local regulations.

CAUTION

Do not use to view the sun directly. Filter category 03. The sun emits energy in the UVA and UVB parts of the solar spectrum that can damage the eyes. The lenses in this product offer protection from this harmful radiation.

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